Slides for Professional Testing Model

Seldom, if ever before, in the history of vision testing has such a high degree of accuracy been achieved in the preparation of test slides. The painstaking selection of the subject material, its precise calibration and accurate photographic reproduction assure standards of test-retest reliability, quality, stability and permanence.

Three sections of tests are provided with the Professional Model Vision Tester.

- 1. The Michigan Pre-School Test slides provide an effective and unique approach in testing visual acuity of children from 3½ years of age through grade 1.
- 2. The Massachusetts Vision Test can be administered quickly with dependable, valid results to elementary school pupils as well as older children.
- 3. The Adult Series provide an excellent method of screening secondary school children and adults. This group has been approved by the FAA for aeromedical examinations.

The order in which the slides are arranged was carefully designed to: (1) test the most basic function, visual acuity, (2) put the subject at ease by starting with an easily understood test and (3) provide clues to the administrator as to difficulties the subject might encounter at later stages in the test battery.

Scoring the Record Form

(1) Print subject's full name and address.

(2) Indicate whether the subject is wearing glasses or contact lenses.

(3) Give subject age in years.

(4) Fill in the date.

Group I—Pre-School Test:

Simply insert a check in the appropriate column (PASS or FAIL) for the individual test.

Group II—Elementary Grades:

Acuity FAR—Record the specific acuity level. Example: If subject was able to read line D with the right eye he would demonstrate a 20/20 in the test result column on the record form. If when you tested the left eye he was only able to pass line B, the subject would be demonstrating a 20/40 acuity. You would record 20/40 in the test result column.

Excessive Farsightedness—Record a check in the appropriate PASS or FAIL column.

Muscle Balance—Record a check in the appropriate PASS or FAIL column.

Color Perception—Record a check in the appropriate PASS or FAIL column.

Group III—Adult:

Acuity FAR-Record the specific acuity level indicated by the test. Example: If the subject passes line 7 of the HK column he has demonstrate 20/20 acuity for his right eye. You would record 20/20 as shown on the record form. The same procedure is used for the left eye using the ZN column, and both eyes using the RO column.

Color Perception—Record a check in the appropriate column.

Phoria—Circle the appropriate number to indicate the subject's response.

Acuity NEAR-Use the same procedure as indicated for FAR acuity.

RECORD FORM for TITMUS VISION TESTER—PROFESSIONAL MODEL

	NAME			DATE			
	ADDRESS		RIG	HT EYE $\frac{20}{200}$ $\frac{20}{100}$ $\frac{20}{70}$ $\frac{20}{50}$ $\frac{20}{40}$ $\frac{20}{30}$	20 20 LEFT EYE 20 200	20 20 20	<u>20 20 20</u>
	ADDRESS						40 30 20 E ∃ Ш
	· .			LINE 2 3 4 5	6 LINE	1 2 3	4 5 6
				20/50 (Blue) 20/40 A W W 3 M			∃ W m
	WEARING GLASSES YES NO			20/40 8 W W 3 M W (Red) C 3 M E E 3			3 3 E
	CONTACT LENSES YES NO			20/20	(, , , , , , , , , , , , , , , , , , ,		m a E
	AGE			(Green)	(Green)	C 3 W	3
-	•.						
	PRE-SCHOOL TEST						
	20/20, 20/30, 20/40, 20/50, 20/70, 20/100						
	(Circle Acuity Level at which You are Testing) Must Read All Pictures Correctly LENS LEVER AT FAR TO WHICH PICTURE DO TABLE LEGS POINT?						
	RIGHT EYE TEST				YE TEST		
	Left Switch at OFF DIAL AT #2 BIRD		FAIL	Right Swite DIAL AT #4	BOY	PASS	FAIL
	DIAL AT #3 RABBIT		†	DIAL AT #4	GIRL		
	DIAL AT #4 BOY		<u> </u>	DIAL AT #2	RABBIT		<u> </u>
	DIAL AT #1 BINOCULARITY SLIDI	 E		BOTH SWITCHES ON	3 W E	<u> </u>	
		. =				1	
ELEMENTARY GRADE TESTS							
	ACUITY - FAR POINT • LENS LEVER AT FAR 4 OUT OF 6 = PASS TEST RESULT						
	DIAL AT #5 RIGHT EYE	OFF	20,	/			
ı	DIAL AT #5 LEFT EYE	OFF	20,	/			
TEST FOR EXCESSIVE FARSIGHTEDNESS • PLUS LENS • FAR POINT (Insert						PASS	FAIL
	DIAL AT #5 LEFT EYE			RIGHT EYE SWITCH			
	DIAL AT #5 RIGHT EYE LEFT EYE SWITCH OFF						
ł	(WITHDRAW PLUS LENS UNIT) A READING OF 20/30 OR 20/20 LINE THROUGH PLUS LENS = FAIL MUSCLE BALANCE TEST — FAR POINT ● LENS LEVER AT FAR PASS FAIL						
ł) X)	PASS	FAIL
ı	(1) DIAL AT #6 LEFT EYE SWITCH OFF (EXPLAIN SANDBOX) (2) DIAL AT #6 RIGHT & LEFT SWITCH ON (RED BALL IN SANDBOX = PASS)						
ı	MUSCLE BALANCE TEST - NEAR POINT . LENS LEVER AT NEAR						FAIL
	DIAL AT #8 BOTH SWITCHES REMAIN AT ON (RED BALL STILL IN SANDBOX=PASS)						
I	SECONDARY SCHOOL AND ADULT TEST						
ŀ							
1	DIAL AT #9 ACUITY FAR RIGHT E	LEFT EYE 20/	YE 20/ BOTH EYES 20/				
	DIAL AT #7 COLOR DEDCEDTION	AD 1-		28 44 58 48 5		PASS	FAIL
ł	DIAL AT #7 COLOR PERCEPTION FAR 1 = 2 = 3 m 4 w 5 m 6 = 7 w 8 = DIAL AT #10 LATERAL PHORIA FAR 1 / 2 / 3 / 4 5 6 7 8 9 10 11 12 13 / 3 / 4 / 5 / 8 / 8 / 9 10 11 12 13 / 3 / 4 / 8 / 8 / 8 / 8 / 8 / 8 / 8 / 8 / 8						
Ì							
I	DIAL AT #12 ACUITY NEAR RIGHT E	U. J. Hilliam	<u>/////// 3</u>	LEFT EYE 20/	BOTH EYES 20	/	
L			D. /				
	DIAL AT #9 VISUAL ACUITY TESTS LEFT EYE BOTH EYES RIGHT EY		אוט	ALAT#12 VISUAL	BOTH EYES RIGHT EYE	AT NE	AR
	1 ZN RO HK	20/20	0	1 s v c	NRK HZO	20/1	100
	2 RKS HNC ZOD			2 RNZH	DOKV CSZN	20/	
	3 HCDV SKZO RNDS			3 CKVD	SNZR DOHC	20/5	
	4 ZROD NSCH VZKN			} 		4	
				4 VHRN	ODSK NZCS	20/4	
	5 KHSC OZNR DNVC			5 HSKRC	NZDOV Z SHNK	20/3	
	6 ONRZV DKHCS KDSON			6 ZONVR	HCSKD VKCDS	20/2	20
	7 SDCHN VRZKO HSNRI	D 20/20					